



Bridges of Hope Training (Pty) Ltd











Inspiring Healthy Lives

Tel: +27 21 790 3245 Cell/WhatsApp: +27 83 539 9181

E-mail: peter@boht.org (Peter Labouchere)

Website: www.boht.org

Training Kits, Models & Materials

Product	Product
<p>1. E-Shaft</p> <ul style="list-style-type: none"> • Condom training model with ejaculation feature • Brown or Blue • 18cm long 	<p>5. Tailored Training Kits & Facilitator Training Workshops</p> 
<p>2. O-Cube</p> <ul style="list-style-type: none"> • Receptive / Female condom training model 	<p>6. The Complete Condom Training Kit: E-Shaft / O-Cube</p> <ul style="list-style-type: none"> • Brown or Blue E-Shaft • O-Cube • Pot of liquid 'semen' • Facilitators guide with extra activities 
<p>3. V-Model</p> <ul style="list-style-type: none"> • Vulva / receptive training  <p style="text-align: right;">model</p>	<p>7. The Complete Condom Training Kit: E-Shaft / V-Model</p> <ul style="list-style-type: none"> • Brown or Blue E-Shaft • V-Model • Pot of liquid 'semen' • Facilitators guide with extra activities 
<p>4. VMMC Training Kit</p> <ul style="list-style-type: none"> • Model with retractable • Facilitators with extra to explain, objections, and VMMC • Patent No. 2012/08354  <p>2-colour foreskin Guide activities address motivate</p>	<p>8. MASKS</p> <div style="display: flex; justify-content: space-around;"> <div data-bbox="938 1659 1050 1832"> <p>3-ply masks</p>  <ul style="list-style-type: none"> • Prevent glasses fogging up by blocking the gaps either side of the nose with a down in flexible wire. • Comfortable • Machine washable / reusable • In a ziplock bag, with instructions • Various fabric colours / designs </div> <div data-bbox="1225 1659 1337 1832"> <p>Buff masks</p>  </div> </div> 

Courier delivery worldwide

Enquiries & Quotations: Contact Peter Labouchere (as above)

Condom Training Kits & Models

1. E-Shaft

- BROWN or BLUE (blue for ethnically diverse groups, or where a less realistic colour is more acceptable)
- 18cm long, "FOR EXTERNAL USE ONLY" on base
- "Ejaculates" (optional) by pressing the base of the E-Shaft, using water or a synthetic cream. This clarifies the importance of expelling air from the condom teat, and enables practice removing and disposing of the condom without touching the semen. The waxy surface makes it easy to slide off the male condom.



"Ejaculation" without condom:



"Ejaculation" into the teat of a male condom:



2. O-Cube

- Female / Receptive model for female condom demonstration.
- In combination with the E-Shaft, the O-Cube demonstrates all aspects of both male and female condom use, for vaginal, anal and oral sex.
- Inflated dimensions: Approximately 10cm x 10cm x 10cm



3. V-Model

- Vulva model for female condom demonstration and training
- Latex/Rubber outer section, transparent plastic at the back
- Approximately 12cm long



6. The Complete Condom Training Kit: E-Shaft / O-Cube for Demonstrating, Practicing, Negotiating and Motivating Male and Female Condom use

“An absolutely wonderful innovation”

Denise van Dijk, Director of Global Market Development
The Female Health Company

- **E-shaft** ‘ejaculating’ model (brown or blue - see page)
- **O-Cube** female / receptive model
- Male & female condoms
- Pot of synthetic cream to represent semen
- **Facilitator’s Guide** - 16 page colour illustrated booklet with participatory training activities to address objections and build skills to negotiate and use male and female condoms.
- **Cylindrical container** (with client’s logo for 20+ units)



Simple to fill the E-Shaft with water / synthetic cream



“Ejaculation” inside the O-Cube



Showing how to hold the base of the male condom when withdrawing



Inserting a female condom



Guiding the penis in (blue E-Shaft model)

“The Bridges of Hope practical exercises and activities, adapted and incorporated into the national condom training curriculum, have enhanced the sessions by making the lessons more interactive and participatory.”

Garika Chidarikire, National Training Coordinator, Department of Health - South Africa

7. The Complete Condom Training Kit: E-Shaft / V-Model

- **E-shaft** 'ejaculating' model (brown or blue)
- **V-Model** female / receptive model
- Male & female condoms
- Pot of synthetic cream to represent semen
- **Facilitator's Guide** - 20 page colour illustrated booklet with participatory training activities to address objections and build skills to negotiate and use male and female condoms.
- Box container (with client's logo for 20+ units)



4. VMMC Training Kit

Patent no: 2012/08354



Uncircumcised



Retracting the foreskin



Fully retracted, large area of HIV-susceptible inner foreskin exposed



Circumcised, much reduced HIV-susceptible area



VMMC Training Kit

The VMMC Training Kit includes:

- VMMC Model
- Facilitators Guide – Colour illustrated instruction booklet
- Participatory activity for motivating VMMC and overcoming objections and reservations.
- Activity for clarifying and motivating 6 weeks abstinence
- Cylindrical container (with client's logo / branding for orders over 20 units)

8. MASKS


ANTI-FOGGING FABRIC MASKS

3-ply masks




- Prevent glasses fogging up by blocking the gaps either side of the nose with a sewn-in flexible wire.
- Comfortable
- Machine washable / reusable
- In a ziplock bag, with instructions
- Various fabric colours / designs:

Buff masks

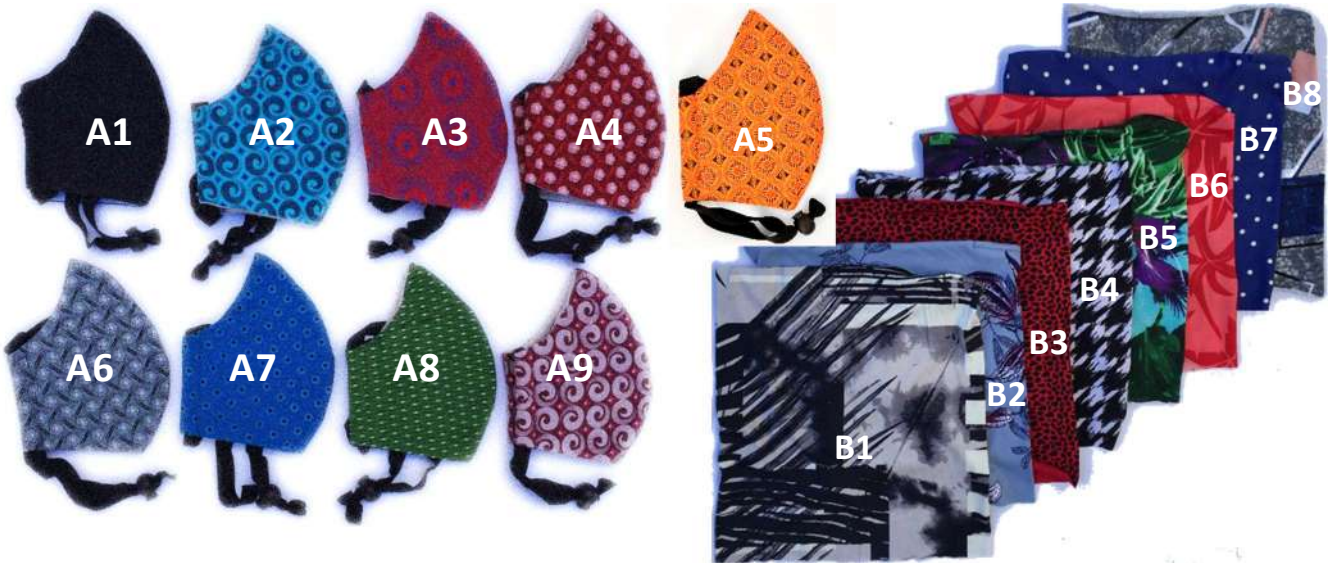

Extra Features of the 3-ply Mask:

- Toggle to adjust the size and tightness
- Soft, broad elastic behind the head
- Sizes: Large/Adult (A) & Small/Child (C)
- Highly efficient filtration:

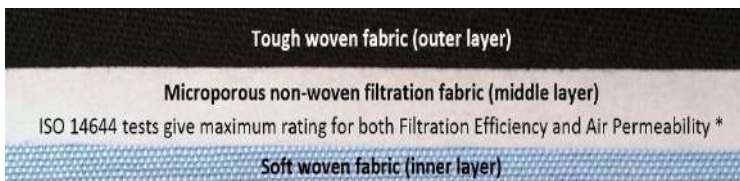

* Technical Data Sheet provided on request